

## REMEDY® RADEL® KNEE SPACER TRIAL

**The REMEDY® Radel® Knee Spacer Trial** is a PPSU device composed of two independent articulating components (REMEDY® Radel® Femoral Trials, available in four sizes and the REMEDY® Radel® Tibial Trial – available in three sizes) that must be combined to fit the anatomy of the patient. A REMEDY® Radel® Tibial Insert Wedge Trial is also available and comes in 3 different sizes.

Each REMEDY® Radel® Femoral Trial is matchable to each REMEDY® Radel® Tibial Trial, while each REMEDY® Radel® Tibial Insert Wedge Trial is matchable only with its corresponding REMEDY® Radel® Tibial Trial.

The correct combination of the REMEDY® Radel® Knee Spacer Trial enables the surgeon to select the appropriately sized REMEDY® Knee Spacer to be implanted.

**Step 1:** Choose the REMEDY® Radel® Femoral Trial and REMEDY® Radel® Tibial Trial based on the dimension of the removed prosthesis. In case of a large tibial defect, include a REMEDY® Radel® Tibial Insert Wedge Trial.

**Step 2:** Test the device in the patient to determine if the prosthesis is anatomically correct and stable.

**Step 3:** Once the device has been tested and verified, remove it from the patient, and use it as a reference to prepare the REMEDY® Knee Spacer device which will be implanted.